

# INSTALL GUIDE

## NEW CareLink™ Personal uploader

Upload for greater insights, better collaboration and improved control.

**FOR GREATER INSIGHTS AND MORE INFORMED DECISIONS, UPLOAD TO CARELINK™**



Understand your glucose patterns



Upload regularly



Collaborate with your diabetes healthcare team

## GET STARTED BY DOWNLOADING THE NEW UPLOADER INSTALL FILE


A one-time download per computer where you wish to upload your device.



- 1 Log in to CareLink™ Personal
  - Click "How to install Uploader"



- 2 Click the download link and select a location to save the Uploader file

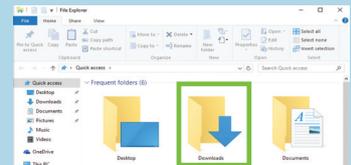


- 3 Find the downloaded file. Open the file and select "Run". You may require administrator rights to your computer.


## AFTER DOWNLOADING, INSTALL THE UPLOADER




- 1 Choose your language

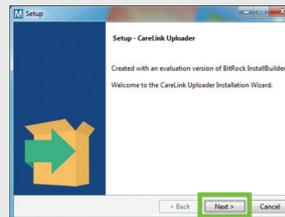

- 2 Click "Next"

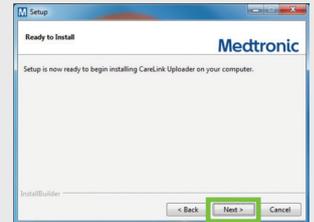

- 3 Click "Next"

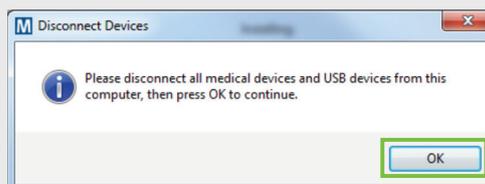

- 4
  - Make sure your CareLink™ USB or meter are not plugged in and click "OK"
  - Click "Finish" to complete the installation

## NOW, YOU'RE READY TO UPLOAD

Return to your CareLink™ Personal Homepage as you are ready to upload your Pump or Meter to CareLink™.

Please turn over for upload instructions.

UC201812041 EN

Medtronic

# UPLOAD GUIDE

**NEW CareLink™ Personal uploader**  
Your first step to understanding more about your diabetes.

## INITIAL FOUR-STEP PROCESS TO ADD A NEW PUMP OR METER



**1** Choose pump or meter


### Pump



**2** Select the pump model



**3** Enter pump serial number

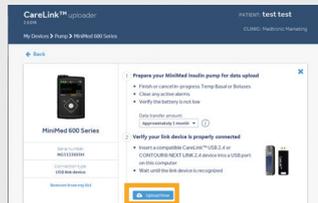

**4**

- Make sure the pump is ready
- Click "Add Pump"

### Meter



**2** Select the meter brand if you want to upload BG meter data only



**3** Select meter model



**4**

- Make sure the meter is ready
- Click "Add Meter"

## TO ENSURE SUCCESS, GET EVERYTHING READY

Plug your USB or meter into your computer

- Place your pump as close as possible to the CareLink™ USB or meter that is plugged into your computer
- Keep the meter linked with the pump




Make sure your pump is ready to upload  
Check that the pump is not on one of these screens.



- No temp basal
- No alarms
- No bolus active
- No low battery


## TWO EASY STEPS TO UPLOAD YOUR PUMP OR METER DATA

### STEP 1

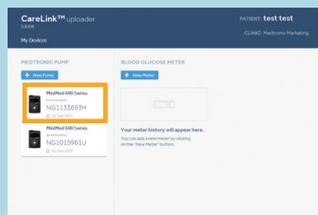

- Select the device you want to upload

### STEP 1

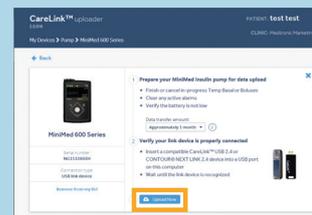

- Make sure the device is ready to upload
- Click "Upload Now"

## LINK WITH YOUR DIABETES HEALTHCARE TEAM



Provide your diabetes team access to your data to save time uploading when you are in clinic. This will help you collaborate with your team for a more informed conversation about your diabetes management.

### Safety Information: CareLink™ software

CareLink™ software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself.

Medtronic Limited  
Building 9, Croxley Park  
Watford  
Hertfordshire  
WD18 8WW

Diabetes Helpline: +44 (0)1923 205167  
medtronic-diabetes.co.uk

Medtronic Ireland Limited  
Block 3090-3094, Lake Drive  
CityWest Business Campus  
Dublin  
DN24 XN47

Diabetes Helpline: +353 (0) 15111444  
medtronic-diabetes.ie

© 2018 Medtronic. All rights reserved. MiniMed, CareLink, Medtronic, the Medtronic logo and Further, Together are trademarks of Medtronic. All other brands and trademarks belong to their respective owners.

UC201812041 EN