


MEDTRONIC CARELINK™ PERSONAL

GENERATING YOUR CARELINK™ REPORTS

Generating reports to gain personalised visual insights into your diabetes therapy is made simple and easy with CareLink™ Personal Software.

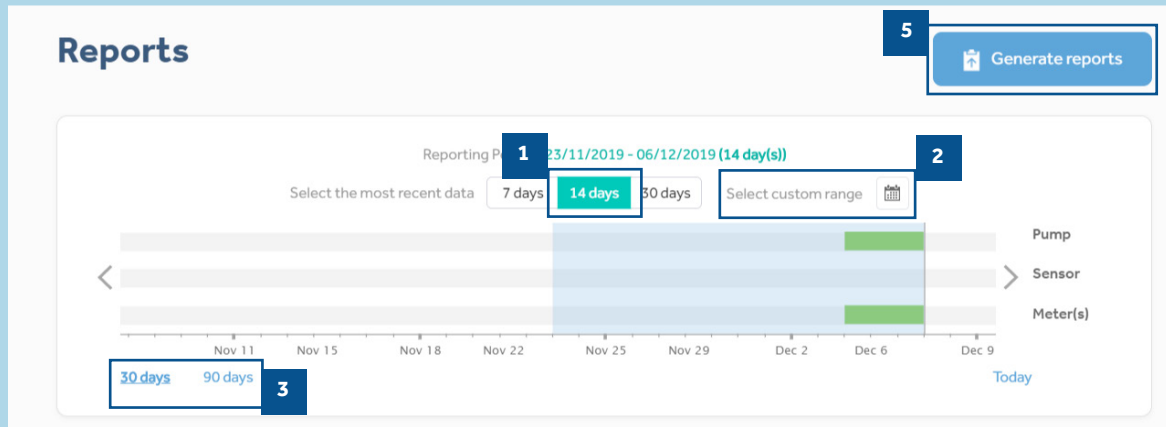

- 1 Time selection.** How many days do you want to look back on?
- 2** Or you can select a **custom date range.**
- 3** Quickly select the **last 30 or 90 days** worth of data
- 4** Select your reports
- 5** **Generate** your CareLink™ reports

4 Select favorites

ASSESSMENT & PROGRESS ☆

Assess glucose control for progress and compare between two time periods.

Choose compare period Not selected

DAILY REPORT ☆

Trouble-shoot problems or copy your best results by reviewing individual days.

Choose days 0 day(s) selected

WEEKLY REVIEW ☆

Review glucose, insulin, food, and events day by day for a week at a time.

DASHBOARD ☆

Visually spot glucose trends and patterns.

MEAL BOLUS WIZARD ☆

Compare BG and SG differences before and after a bolus was given for each meal.

ADHERENCE ☆

Identify better behaviors by tracking how you use your pump.

SENSOR & METER OVERVIEW ☆

Visually spot sensor and blood glucose trends over each hour or meal of the day. Identify relationships between glucose, insulin, food, and events day by day.

LOGBOOK ☆

Compare BG and pump records over days and times in a logbook format.

DEVICE SETTINGS SNAPSHOT ☆

Review and record all of your latest pump settings for safe-keeping.

EPISODE SUMMARY ☆

Review summary of glucose patterns and description of events preceding episodes of low and high glucose.

Data Export (CSV)

Select the reports you wish to generate from the 'report buttons' (you can click on the 'i' icon for an overview of each report, or on the button to select it)

Note: Some reports are only available for specific devices. This will be stated on the report tab.

For your favourite reports, click on the 'star' next to the report name to add it to your favourite list for quick report generation in future.