


# 3 STEPS TO SUCCESS

## GUARDIAN™ SENSOR 3

### 1 START YOUR SENSOR WHEN YOU START YOUR DAY

*Always have your blood glucose (BG) meter nearby on the first day. You may receive more calibration requests on the first day.*

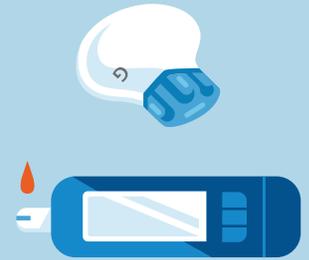

**BE READY TO CALIBRATE:** SENSOR WARM-UP PERIOD MAY VARY FROM 40 MIN UP TO 120 MIN




### CALIBRATE BEFORE MEALS & AT BEDTIME 2

*Calibrate your sensor with the blood glucose (BG) you take for meals before delivering of a meal bolus. Calibrate before bedtime to help avoid nighttime alerts.*

### 3 FOLLOW PUMP INSTRUCTIONS

*BG required alerts are for Auto Mode.  
Calibration required alerts are for continuation of sensor data.  
Just remember to read and follow the pump screens.*

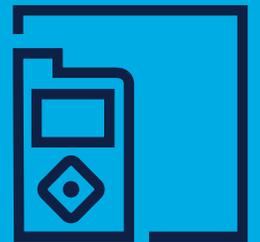

For more tips and tricks, refer to back of this document ►

**Medtronic**

# 3 FREQUENTLY ASKED QUESTIONS ABOUT GUARDIAN™ SENSOR 3

## Q1: When should I calibrate?

We recommend that you calibrate your sensor with the BG you take for meals before delivery of a meal bolus, and before going to bed. Spreading out your sensor calibrations over the day will help optimise sensor performance and minimise alerts. You can calibrate anytime. However, calibrating with two or three trend arrows may temporarily decrease accuracy until the next calibration. Therefore we recommend to calibrate when your glucose is stable and you will not see two or three trend arrows on the display.

## Q2: How long does my sensor last?

While the sensor lasts up to 168 hours (7 days), if your sensor is not performing well, you may receive a Change Sensor alert. This is to ensure you are only provided with safe, reliable and accurate sensor glucose values.

## Q3: What is the difference between BG required and Calibration required alerts?

You are prompted for a BG required alert to stay in, or enter Auto Mode. You are prompted for a Calibration required alert to make sure you continue to receive accurate and reliable Sensor Glucose (SG) values. If you get a BG required alert to enter or stay in Auto Mode, and it's time to calibrate, go ahead and use the BG for a calibration.

**KEEP IN MIND:** *If a BG required alert occurs after a BG is entered, it means the first BG was not accepted by Auto Mode. It may help to wait 30 minutes before re-checking and entering the second BG.*

## Have More Questions? Contact our 24-hour HelpLine 01923 205167

For a listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use.

Medtronic Limited  
Building 9, Croxley Park  
Watford  
Hertfordshire  
WD18 8WW  
Diabetes Helpline: +44 (0)1923 205167  
medtronic-diabetes.co.uk

Medtronic Ireland Limited  
Block 3090-3094, Lake Drive  
CityWest Business Campus  
Dublin  
DN24 XN47  
Diabetes Helpline: +353 (0) 15111444  
medtronic-diabetes.ie